


# PURINOX HME<sup>®</sup>

Instructions for use

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## Figures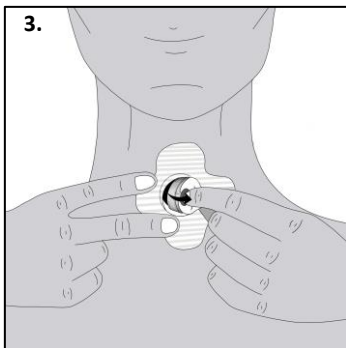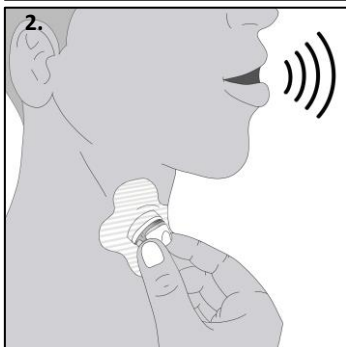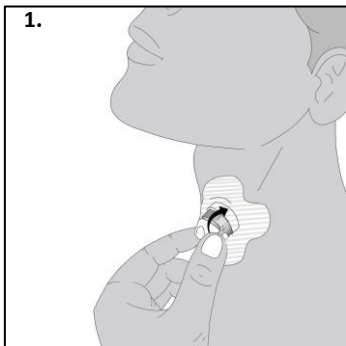

## **Disclaimer**

Purinox HME B.V. makes no express or implied warranty to the purchaser hereunder with regard to the lifetime of the delivered product, which may vary according to individual use and biological conditions. In addition, Purinox HME B.V. makes no warranty as to the merchantability or fitness of the product for any particular purpose.

## **Content**

EN - English .....	5
NL - Nederlands .....	12

# EN - English

## 1 Descriptive information

### 1.1 Intended use

The PURINOX HME® is a 22 mm reusable passive heat and moisture exchanger for patients breathing through a tracheostoma. The PURINOX HME® is intended for single-patient use and for adults only.

### 1.2 Contra-indications

The device should not be used by patients with impaired cognitive or physical abilities. The device should not be used by patients who are unable to attach or remove the device themselves, or who do not have sufficient knowledge of how to use the device, or who are not cognitively able to understand the risks associated with its use.

The device should not be used by patients with low resting tidal volume, as the additional dead volume may cause retention of CO<sub>2</sub> (carbon dioxide).

### 1.3 Device description

The PURINOX HME® is suitable for the majority of situations because of the balance between humidification and breathing resistance. The device is reusable after cleaning for 30 cleaning cycles or a maximum of 3 months after first use.

The PURINOX HME® is affixed over the tracheostoma on an tracheostomy adhesive with a 22 mm coupling ring or a tracheostomy tube with a 22 mm connector. A PURINOX HME® cover can be clicked onto the PURINOX HME® to adjust the appearance.

### 1.4 Technical data

	Moisture loss at V <sub>T</sub> = 1000 ml* (mg/l)	Pressure drop (hPa)			Internal volume (ml)*
		at 30 l/min*	at 60 l/min*	at 90 l/min*	
PURINOX HME®	16.7	0.4	1.2	2.4	4.9

\* According to ISO 9360

Material PURINOX HME®: stainless steel 316L

Material PURINOX HME® cover: high density polyethylene

## **1.5 Content of the sets**

### **Starter set**

- 4 PURINOX HME®
- 4 PURINOX HME® cover
- 4 Storage container
- 1 Ultrasonic cleaner
- 60 Steradent® tablets (Triple Action Plus or Active Plus)
- 6 TePe® Interdental Brush Orange
- 1 Return envelope

### **Replacement set**

- 4 PURINOX HME®
- 4 Storage container
- 60 Steradent® tablets (Triple Action Plus or Active Plus)
- 6 TePe® Interdental Brush Orange
- 1 Return envelope

### **Replacement set with covers**

- 4 PURINOX HME®
- 4 PURINOX HME® cover
- 4 Storage container
- 60 Steradent® tablets (Triple Action Plus or Active Plus)
- 6 TePe® Interdental Brush Orange
- 1 Return envelope

## **1.6 WARNINGS**

- Do not apply unintended force on the PURINOX HME®. Accidental closure of the device may obstruct breathing.
- Do not apply excessive force while attaching the PURINOX HME® or while it is attached. Excessive force may result in the device being pushed through the coupling ring of the adhesive. This can obstruct breathing and block the airways.
- Do not use the PURINOX HME® with any other attachment devices than advised. The device may go through the coupling ring of the adhesive or get stuck in the attachment device. This can obstruct breathing and block the airways.
- Do not attach a PURINOX HME® that is clogged with mucus as this may obstruct breathing.
- Do not reuse the PURINOX HME® on multiple patients. This may cause cross-contamination. The device is intended for single-patient use only.
- Do not use incompatible or non-genuine devices as this could result in personal injury or damage to the device.

- Do not use the PURINOX HME® during MRI examination. The device contains metal and will interact with the MRI field.
- Consult your physician if you have ANY adverse reaction to the use of the PURINOX HME®.

## **1.6 PRECAUTIONS**

- Always check if the PURINOX HME® is clean and undamaged prior to use.
- Always clean the PURINOX HME® according to the instructions below. If the device is not properly cleaned, it may reduce function and cause infection.
- Do not attach a PURINOX HME® that appears slippery due to mucus or other substances, as it will be difficult to attach in a proper position.
- Be careful when attaching the PURINOX HME® at night, as dark surroundings may make it more difficult to attach it in the proper position. It may also be hard to detect if the device is contaminated.
- Do not administer medicated nebulizer treatment over the PURINOX HME® since the medication can become deposited in the device
- Do not use humidifiers or heated humidified oxygen tracheostomy while using the PURINOX HME®. The device will become too wet. If oxygen therapy is required, use non-heated humidified oxygen.
- Do not use the PURINOX HME® while showering or during water activities since water may enter the airways.
- Be careful using the device in extremely hot (e.g. sauna, long sun exposure time) or cold environments. The PURINOX HME® functions best between 36 and 94°F (2 and 34°C).
- Be careful using the PURINOX HME® during extreme activities (e.g., contact sports, motorcycle riding) since it can cause unexpected force onto the device or extreme airflow through the device.

## **2 Instructions for use**

### **2.1 Operation instruction**

#### **2.1.1 PURINOX HME®**

##### **Preparation**

Inspect if the PURINOX HME® is clean and undamaged. Place the attachment device, according to its instructions for use.

### **To attach**

Hold the PURINOX HME® at the side openings between your thumb and index finger. Press lightly until the device connects to the attachment device. Rotate the device such that the airflow holes can easily be sealed with thumb and index finger. See fig. 1.

### **To speak**

Seal the two side openings of the PURINOX HME® with thumb and index finger at the same time to allow speech. See fig. 2.

### **To remove**

Hold the attachment device. Place the index finger in one side opening and the thumb onto the opposite side opening of the PURINOX HME®. Pull lightly sideways until the device disconnects. See fig. 3.

## **2.1.2 PURINOX HME® cover**

### **To attach**

Click the PURINOX HME® cover onto the device preferably before attaching the device is not affixed over the stoma.

### **To remove**

Removing the PURINOX HME® cover is easiest when the device is not affixed over the stoma. Remove the PURINOX HME® cover by placing your finger behind the indentation and pulling lightly sideways until the cover disconnects from the device.

## **2.2 Cleaning**

### **2.2.1 PURINOX HME®**

The PURINOX HME® should be cleaned at least after 24 hours of use according to the method below to keep good function and prevent infection. The device is cleaned easiest immediately after use. The PURINOX HME® can be safely cleaned for 30 cleaning cycles. Make sure to alternately use and clean the four PURINOX HME®s of a set.

The PURINOX HME® may be rinsed with water and dried, if necessary, in between performing the cleaning method as described below.

**Caution:** Do not clean in any other way than described below.

Remove the cover from the PURINOX HME® and wash your hands before cleaning.

### **Step 1: Rinse**

- a. Rinse the PURINOX HME® under running lukewarm (68-104°F / 20-40°C) drinking water. Keep each of the three openings under the water stream for at least 10 seconds.
- b. If any PURINOX HME® channels remain clogged, use a clean TePe® Interdental Brush Orange to clean these channels and rinse the device again according to step 1a.

**Caution:** Always rinse the device after using the brushes to avoid any residue in the channels.

### **Step 2: Ultrasonic cleaning**

Use an ultrasonic cleaner with the following specifications: frequency 40-45kHz, power 18W and maximum volume 500ml. Preferably use the ultrasonic cleaner from the starter set. Use an ultrasonic cleaner which is only used for cleaning PURINOX HME®s. Always follow the instructions for use of the ultrasonic cleaner.

- a. Fill the ultrasonic cleaner with lukewarm (68-104°F / 20-40°C) drinking water. The water should not exceed the maximum line but should be enough to place the PURINOX HME® fully underwater.
- b. Place a maximum of 4 PURINOX HME®s in the ultrasonic cleaner. Shake the devices to remove all air bubbles from the channels. Position the PURINOX HME®s sideways.

**Caution:** Do not place more than 4 PURINOX HME®s in the ultrasonic cleaner.

- c. Add a Steradent® tablet (Triple Acton Plus or Active Plus variant) to the water and wait for 3 minutes.

**Caution:** Do not use other variants of Steradent® tablets.

**Caution:** Always follow instructions for use of Steradent® tablets.

- d. Start the ultrasonic cleaner and let it run for 3 minutes. The ultrasonic cleaner from the starter set automatically stops after 3 minutes.

### **Step 3: Rinse**

- a. Rinse the PURINOX HME® under fast running lukewarm (68-104°F / 20-40°C) drinking water. Keep each of the three openings under the water stream for at least 10 seconds.
- b. Empty the ultrasonic cleaner and rinse the cleaner with water.
- c. Inspect the PURINOX HME® regarding mucus and debris residues. Repeat steps 1 and 2 if necessary.

**Caution:** If the PURINOX HME® does not get clean, dispose the device. Do not reuse an unclean device.

#### **Step 4: Dry**

- a. Shake the PURINOX HME® to remove as much water as possible from the channels.
- b. Place the PURINOX HME® on a clean paper towel and leave it to air-dry for at least 2 hours. When the PURINOX HME® is completely dry, it can either be used again or placed in a clean, closed storage container.

**Caution:** Do not use the PURINOX HME® until it is completely dry after cleaning.

#### **2.2.2 PURINOX HME® cover**

The PURINOX HME® cover should be cleaned when contamination is visible or at least weekly. Remove the cover from the PURINOX HME® and wash your hands before cleaning.

1. Soak the cover in a mild solution of liquid dish soap and lukewarm (68-104°F / 20-40°C) water for 10 minutes.
2. Clean the cover with a clean dishwashing brush or soft cloth.
3. Rinse the cover under running lukewarm (68-104°F /20-40°C) water for at least 10 seconds on both sides.
4. Place the cover on a clean paper towel and leave it to air-dry for at least 2 hours.

#### **2.3 Storage instructions**

When not in use, clean the PURINOX HME® and PURINOX HME® cover as described above and then store them in a clean and dry storage container at room temperature.

#### **2.4 Lifetime and disposal**

The PURINOX HME® can be used for 30 cleaning cycles or a maximum of 3 months. If the device shows signs of damage or irremovable contamination, it must be disposed of. The PURINOX HME® can be returned to the manufacturer for safe disposal. Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

The PURINOX HME® cover should be disposed of when showing signs of damage or irremovable contamination. The cover can be disposed of with plastic waste.

### **3 Reporting**

Any serious incident that has occurred in connection with the device must be reported to the manufacturer and the national authority of the country where the user and/or patient resides.

Instruction videos are available on  
[www.youtube.com/@purinoxhme](http://www.youtube.com/@purinoxhme)

View the most recent version of these instructions for  
use on [www.purinoxhme.com](http://www.purinoxhme.com)

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